

The Coseal kit includes:



LIQUID COMPONENTS POUCH consists of two syringes containing solutions, preassembled into delivery housing.




POWDER COMPONENT POUCH consists of a syringe containing two polyethylene glycols, or PEGs in powder form.



APPLICATOR POUCH contains two applicator tips.



1 Remove Luer cap from transfer port closure. Attach powder syringe to transfer port on syringe housing. Ensure syringe clip on syringe housing is rotated away from free plunger. Remove Luer cap from powder syringe.

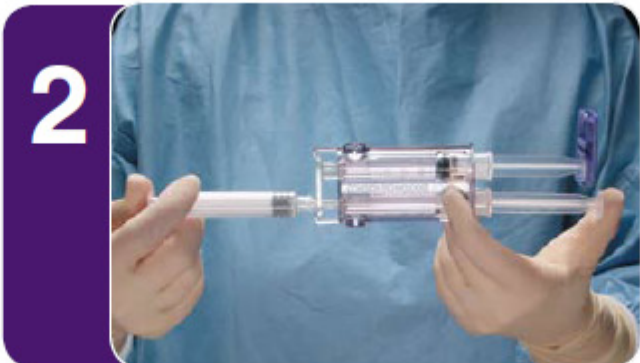

2 Transfer the liquid into the powder syringe by forcefully depressing the free plunger on the syringe housing. Mix contents back and forth until the solid is completely dissolved (at least 20 times). End with entire contents in the syringe housing.



3 Disengage the powder syringe and transfer port together by depressing levers on the side of syringe housing.



4 Holding the syringe tips up, level the plungers and rotate the syringe clip to connect the free plunger.



5 Hold the assembly upright and expel all the air from both syringes.



6 Attach applicator tip onto the end of the syringe housing. Coseal surgical sealant is ready to use. Use Coseal within 2 hours of preparation.

Please refer to package insert for warnings, precautions and instructions for use.



Baxter Healthcare Ltd. Wallingford Road Compton, Newbury RG20 7QW www.baxterhealthcare.co.uk	Baxter Healthcare Ltd. Unit 7, Deansgrange Business Park Blackrock Co.Dublin www.baxterhealthcare.ie
--	--

UKI-AS13-230007 August 2023

Any medical device product quality complaints (including medical device adverse incidents) relating to Baxter products can be reported directly to the Baxter Country Quality Assurance Team: In the UK on +44 (0)1604 704603, or by email to UK_SHS_QA_Complaints@baxter.com. In Ireland on +353 (0)1 2065500 or by email to shs_complaints_dublin@baxter.com

Alternatively please report directly to your Baxter Representative, who will take the details and forward to the Baxter Country Quality Assurance Team.

Medical device adverse incidents should also be reported:

In the UK to the MHRA. Reporting forms and information can be found at: www.mhra.gov.uk/safetyinformation/reportingsafetyproblems/index.htm .

In Ireland to the HPRA. Reporting forms and information can be found at: <http://www.hpra.ie/homepage/about-us/report-an-issue>

Baxter and Coseal are trademarks for Baxter International Inc., its subsidiaries or affiliates.